

# *pedi·padz<sup>®</sup> II*

Pediatric Electrodes for the  
AED Plus<sup>™</sup> and AED Pro<sup>®</sup>



## Expressly Designed for the ZOLL AED Plus

*pedi•padz*<sup>®</sup> II electrodes are specifically designed, tested, and validated for use with the ZOLL AED Plus<sup>™</sup>.

With ZOLL AED Plus and AED Pro<sup>®</sup> electrodes and standard cable connectors, you are assured of a complete system. No need for adapters, special connectors, or modified cables.

There are no compromises with ZOLL electrodes.

ZOLL is the name you can rely on for high performance, proven efficacy, and ease of use.

## The Only Full-Rescue AED with Intelligent Pediatric Capability

The ZOLL AED Plus Intelligent Pediatric Capability includes voice and text prompts that tell rescuers that *pedi•padz* II are connected, ensuring that the proper electrodes are being used. Separate and specific algorithms for adults and children are used to analyze a victim's heart rhythm. Separate energy levels also have been incorporated for pediatric patients. Electronics in the AED Plus adjust defibrillating energy levels automatically so that suitable doses are delivered accurately, when needed.

*pedi•padz* II electrodes are designed especially for use with the AED Plus and AED Pro, and are also compatible with the M Series<sup>®</sup> and E Series<sup>™</sup>, allowing continuity of care\*. The AED Plus and AED Pro do not rely on a resistor or other hardware in the wires to reduce delivered energy. Anatomical diagrams on the package facilitate proper pad placement.



### *pedi•padz* II Specifications

*Shelf Life:* 24 months

*Backing Material:* Closed Cell Polyethylene

*Conductive Gel:* Hydrogel

*Conductive Element:* Tin

*Connection Method:* Pre-attached

*Packaging:* Paper/Polyfoil Laminate

*Impedance Class:* Low

*Length, Lead Wires:* 34.0 inches / 86.4 cm

#### **FRONT (Sternum) Electrode**

*Shape:* Round

*Diameter, Total:* 3.7 inches / 9.4 cm

*Area, Total:* 10.75 square inches / 69.4 sq cm

*Diameter, Conductive Gel:* 2.85 inches / 7.24 cm

*Area, Conductive Gel:* 6.37 square inches / 41.1 sq cm

#### **BACK (Apex) Electrode**

*Shape:* Rectangular

*Length, Total:* 5.43 inches / 13.8 cm

*Width, Total:* 3.5 inches / 8.9 cm

*Area, Total:* 19.0 square inches / 122.6 sq cm

*Length, Conductive Gel:* 3.58 inches / 9.1 cm

*Width, Conductive Gel:* 2.18 inches / 5.5 cm

*Area, Conductive Gel:* 7.81 square inches / 50.3 cm

#### **Order Information**

*Case Part Number:* 8900-0810

*Case Count:* 1 pair

\*With M Series and E Series defibrillators, *pedi•padz* II must be used in manual mode.

**ZOLL Medical Corporation**  
Worldwide Headquarters  
269 Mill Road  
Chelmsford, MA 01824  
978-421-9655

**For worldwide locations, visit:**  
[www.zoll.com/contacts](http://www.zoll.com/contacts)

Specifications subject to change without notice.

© 2006 ZOLL Medical Corporation. All rights reserved.

"Advancing Resuscitation. Today." AED Plus and E Series are trademarks of ZOLL Medical Corporation.

AED Pro, M Series, *pedi•padz* II and ZOLL are registered trademarks of ZOLL Medical Corporation.

ZOLL Medical Corporation 269 Mill Road Chelmsford, MA 01824-4105 USA

9656-0172 Printed in USA 10K 7/06



**ZOLL**<sup>®</sup>  
Advancing Resuscitation. Today.<sup>™</sup>